

DATASHEET

FFP MASKS

PROTECTION AGAINST DUST, MIST & FUMES



AIR SERIES – BI-DIRECTIONAL PROTECTION

FFP2 NR D · Size M/L

 **3100** non valved

FFP2 NR D · Size S

 **3150** non valved

FFP3 NR D · Size M/L

 **3200** non valved

FFP3 NR D · Size S


 **3250** non valved

CHARACTERISTICS



ActivForm®

Automatically fits to the face.
No manual adjustments by the user are necessary.

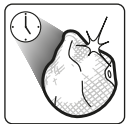


DuraMesh®

Masks have a strong and durable structure.




AirWave® filter

Long-life pleated filter for noticeably reduced breathing resistance.

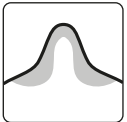


Nose seal

The flexible nose seal improves fit and provides optimum wearing comfort.

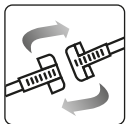


Clip & Adjustable Strap

Easy on & off; mask can be worn around the neck during breaks.

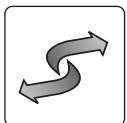

Bi-Directional Protection

Protects the wearer and his surrounding area
Meets the requirements of EN 149 and EN 14683.

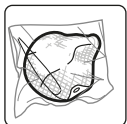

Individually packed

Protected from dust and kept hygienic before use.

**LATEX
FREE**

Latex-Free

Doesn't contain natural latex.

**PVC
FREE**

100% PVC-FREE

All Moldex products and packaging are completely free from PVC.

**S
M/L**

Two sizes

For an optimal fit even for smaller faces.

CERTIFICATION

The Moldex Air FFP-masks meet the requirements of EN 149:2001 + A1:2009. The products are CE-marked in accordance with the requirements of EU regulation (EU)2016/425. The IFA (0121) in St. Augustin (Germany) is responsible for type examination (Module B) and monitoring of production (Module D). In addition to this the unvalved masks of the Air Series meet the requirements of EN 14683:2014 according to the Medical Device Regulation (EU)2017/745. The products are manufactured in an ISO 9001 certified plant.

Information for the UK market: UKCA-marked in accordance with the requirements of PPE Regulation 2016/425 as brought into UK law and amended. Responsible for Module D surveillance: BSI-Assurance UK Ltd. (Identification number 0086), United Kingdom. UK Importer: Moldex/Metric AG & Co. KG · Unit 9, Glaisdale Point · Off Glaisdale Drive, Bilborough · Nottingham. NG8 4GP · Tel. +44 (0)115 985 4288 · info@uk.moldex-europe.com

MATERIALS

Filter Layer, Inner Shell, DuraMesh®: Polypropylene, Ethylene-vinyl acetate (EVA)

Nose Seal, Clip: Polyethylene

Head Strap: Polyester, Lycra

WEIGHT

3100: 26 g **3150:** 21 g **3200:** 28 g **3250:** 23 g

AREAS OF USE

Level	WEL	Hazard type
FFP2	10 x	Hazardous fine dusts, water and oil based mists/aerosols, Biological agents of risk group 2
FFP3	20 x	Harmful and carcinogenic dusts, water and oil based mists/aerosols, biological agents of risk group 2 and 3, CMR-substances

(WEL = Workplace Exposure Limit)

NR (non reusable): Single use.

Comfortable and durable throughout the whole shift.

D

Dolomite clogging test

Masks have passed the Dolomite clogging test, giving the user better breathing resistance for longer.

DATASHEET

FFP MASKS

PROTECTION AGAINST DUST, MIST & FUMES



TESTING ACCORDING TO EN 149:2001 + A1:2009

Total inward leakage

Ten test subjects perform a variety of exercises. During the exercises the amount of test aerosol that penetrates the filter, face seal and valve are sampled. The total inward leakage of 8 out of 10 test subjects shall not exceed the following levels:

Category	max. total inward leakage
FFP2	8 %
FFP3	2 %

The filter penetration after loading the filter with 120 mg paraffin oil according to EN 149:2001+A1:2009 shall not exceed the following levels:

Category	max. Filter penetration
FFP2	6 %
FFP3	1 %

TESTING ACCORDING TO EN 14683:2014

Requirements

Testing	Type I	Type IR	Type II	Type IIR
Bacterial filtration efficiency (BFE) (%)	≥ 95	≥ 95	≥ 98	≥ 98
Splash resistance (mmHg)	not applicable	≥ 120	not applicable	≥ 120

The Air masks pass the examinations of the category IIR.


BFE testing method

An aerosol of *Staphylococcus aureus* is drawn through the filter material. The filter efficiency against bacteria is declared by the amount of colony building units which go through the mask.

INSTRUCTIONS FOR USE

- The masks are available in two sizes. According to the face dimensions the appropriate mask has to be chosen (S or M/L).
- The user has to be trained and instructed in wearing the mask.
- FFP masks do not protect against gases and vapours.
- The oxygen concentration of the ambient atmosphere should be at least 19,5 % Volume.
- These respirators may not be used if the concentration type, and properties of contaminants in the ambient atmosphere are unknown or at dangerous levels.
- Respirators should be disposed if damaged, if the breathing resistance becomes high due to clogging.
- Never tamper with, alter or modify the respirator.

INSTRUCTIONS FOR FITTING

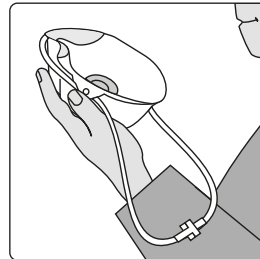


1. Pull strap to form a large loop.



4. Adjust strap by pulling loop on strap.



2. Place respirator on chin and pull loop over head tight to the neck.

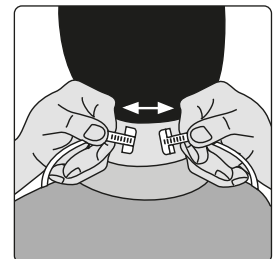

5. During breaks unclip strap.



3. Pull upper strap and place on back of head.



6. Let mask hang around your neck.

INFO

For help on selection and training please contact us. We offer a wide range of training packages and support material.

MOLDEX/METRIC AG & Co. KG
Unit 9, Glaisdale Point
Off Glaisdale Drive, Bilborough
Nottingham. NG8 4GP

Tel.: +44 (0)115 985 4288
Fax: +44 (0)115 985 4211
www.moldex-europe.com
info@uk.moldex-europe.com